

DIY Reversible Close Fitted Mask Project I



SUPPLIES NEEDED

- Sewing Machine
- Scissor
- Ruler
- Pins
- Thread
- Wire (pipe cleaner)
- Two (2) 7" x 9"
Cotton material (100%)
- Two (2) 6 1/2" Elastic



STEP 1

Cut out **two** 7" x 9" cloths of your cotton material.



STEP 2

Measure and cut the following:

- Two (2) 6 1/2" Elastic
- One 6" Wire/Pipe cleaner




STEP 3

- Face measured fabric together (print-side inward and or dull-side out).
- Pin elastic inwards at the wide end of the fabric (edges facing outward). Measure evenly on ends (about 1" from each end/edge).
- After pinning elastics, pin around fabric to secure while sewing.



STEP 4


- Leave room for open at the bottom right (to flip inside out when sewing is complete).

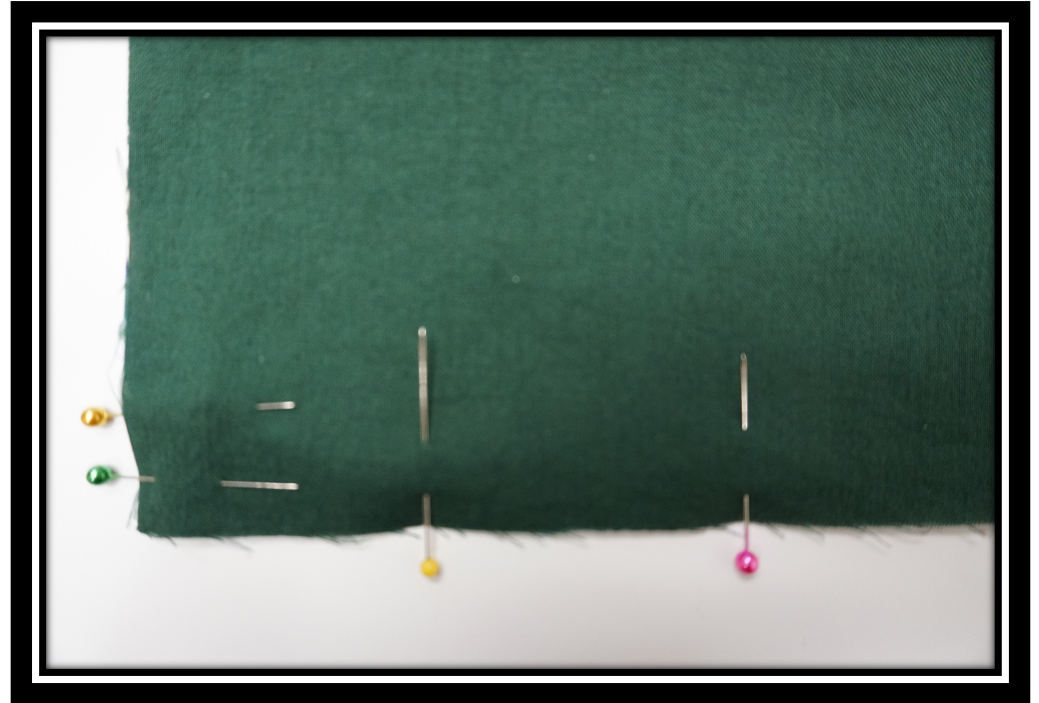

STEP 5


Begin sewing:

- Align edges of fabrics/hems to the end of the presser foot (1/4 inch).
- Sew all around edges making sure to stop at the other end of the opening side.
- Makes sure to **backstitch** at the start and at the end to secure stitching.

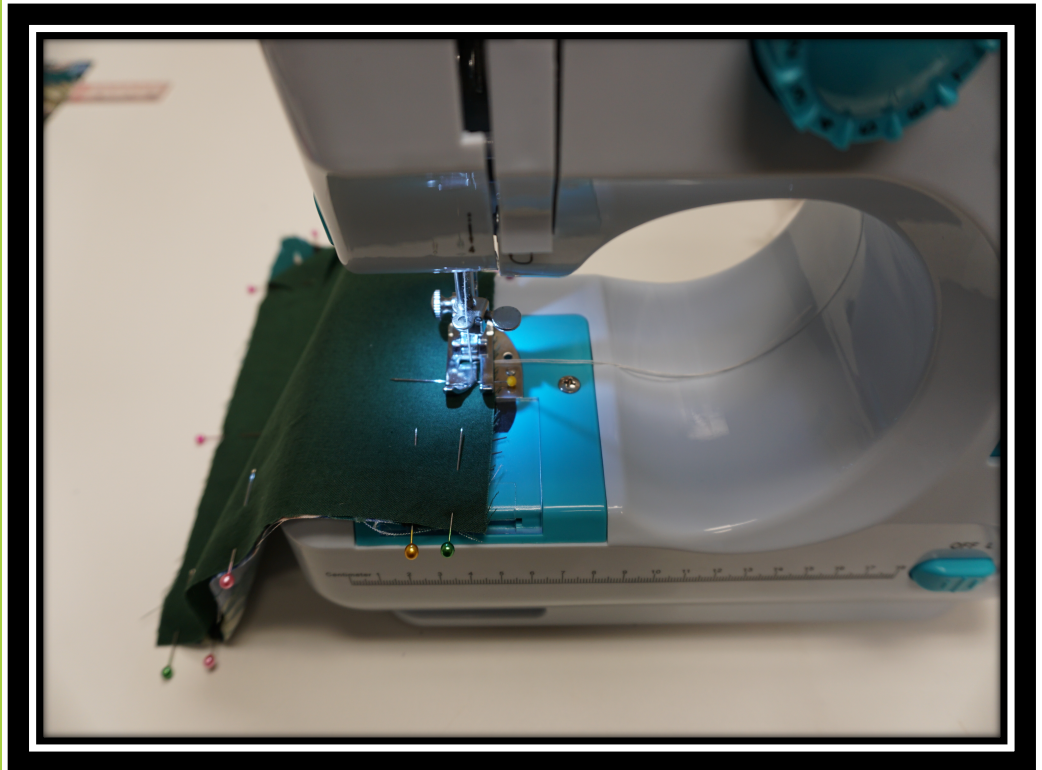

STEP 6

- Once sewing is complete, clip corner, making sure not to cut into the thread.
- Pin 6 inch wire/pipe cleaner to top center, placing wire in center above previous stitching. *Use zig-zag stitch selection and sew across the wire/pipe cleaner.

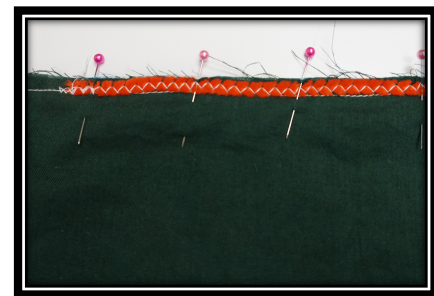

STEP 7

- Flip in-side out.
- Iron if desired.



STEP 8

- Use (3) pins on each side of elastic end.
- Start to make (3) even folds on each side, going down wards (make sure wire is on top end.
- Eyeball for even-ness or about (1 1/4") from each fold.



STEP 9

- Sew sides of mask pleats at both ends.
- Sew/or hand-stitch closing at bottom of mask.
- Easy surgical mask complete!!





ENJOY YOUR REVERSIBLE MASK

Click the link below to see other mask option

